




**BodiMetrics**



## O2 Vibe™ Sleep and Fitness Monitor



# Managing O2 (Me) User Set Up


## Step 1: Setting up Cloud Account - OPTIONAL


- Tap on Me Menu  tab
- Under Cloud heading turn ON Auto Sync by swiping button to the right to turn background **GREEN**
- Create a Cloud Account by selecting **Right Arrow** link
  - Enter your name, email; select a password and confirm
- Tap Sign Up button to complete

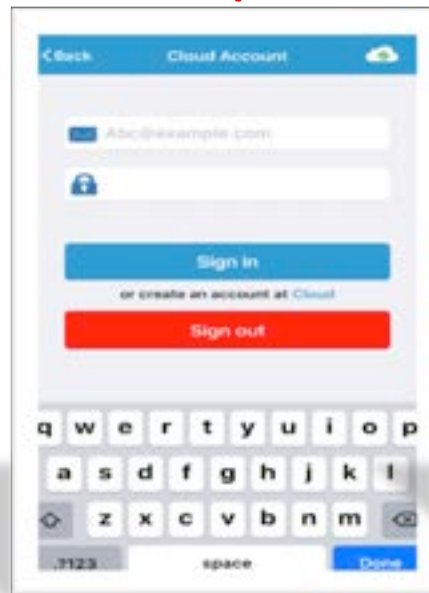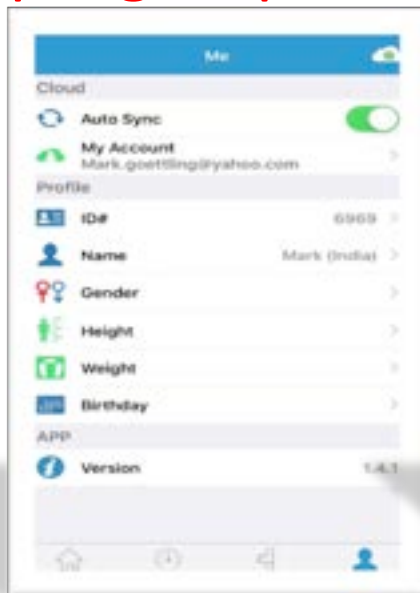

# Managing O2 (Me) User Set Up

## Step 2: Setting up User Info for Cloud Account ONLY

- Tap on Me Menu  tab
- Enter an ID# - any number you select w/1-16 numbers
- Enter your Name and Gender
- Enter your Height and Weight
- Enter your Birthday
- Tap < Back button upper left corner of screen to return to menu

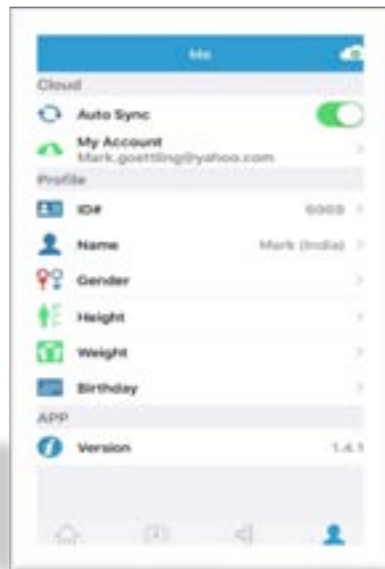



**Health & Wellness in the Palm of Your Hand!**

**Toll Free (844) 744-8800**

**[www.bodimetrics.com](http://www.bodimetrics.com)**