




BodiMetrics

Performance Monitor™

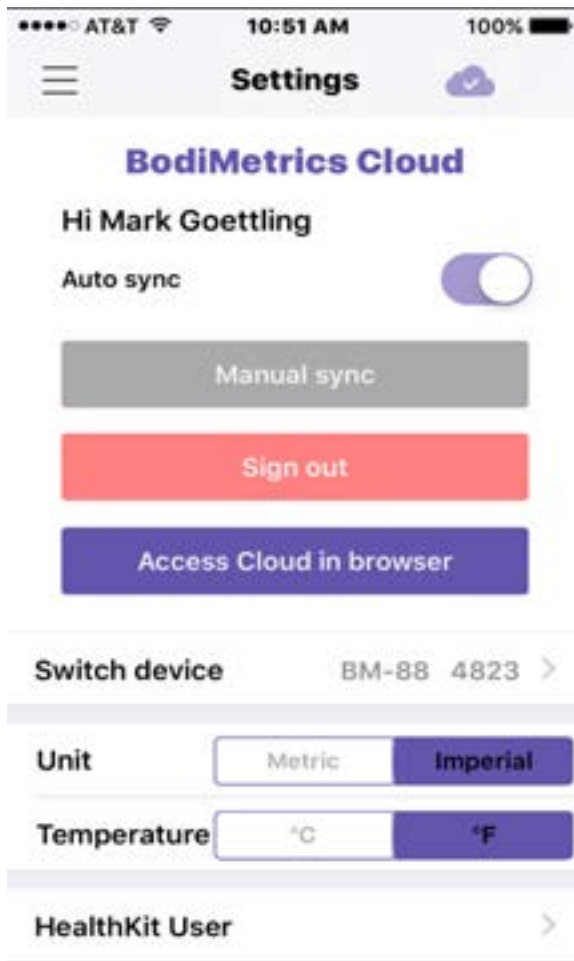




Sending Reports via Email



- Select Report you'd like to email then Tap 
- Next, Tap Share menu
- Then, Tap Email icon and
- Finally, type in the email address and subject line to send your report to a family member, caregiver or physician

Registering User - Cloud



- Tap Main Menu  and Navigate to Settings menu
- Sign into cloud account to sync user data by entering company email and password
- Toggle Auto-sync button to ON 
- Check Units of measure; C° or F° and British or Metric
- Your User is now ready to sync data from their Performance Monitor -> BodiMetrics mobile app -> to your secure cloud Account



Health & Wellness in the Palm of Your Hand!

Mark Goettling (818) 268-6828

mgoettling@bodimetrics.com

Manhattan Beach, CA

www.bodimetrics.com